geemarc™ AMPLICALL™ 110
Hear the Difference

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Français P9

Deutsch P16



Please refer to our website: www.geemarc.com for an up to date user guide, as there may be important updates and changes you need to be aware of.

Box Contents

The AC110 (AMPLICALL™110) box contains the following items:

1 user's manual	2 screws and 2 anchors
1 AC110 (AMPLICALL™110)	2 x AA, 1.5V Alkaline batteries
1 RJ11/RJ11 cable	1 Drill hole template


DESCRIPTION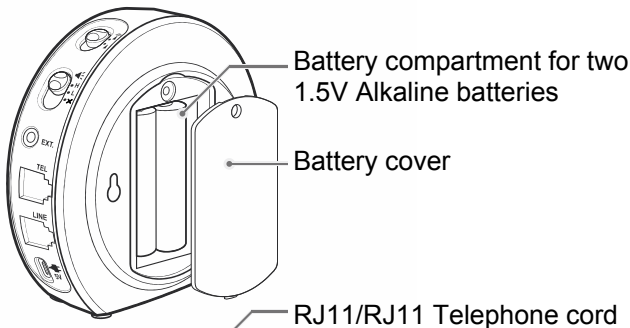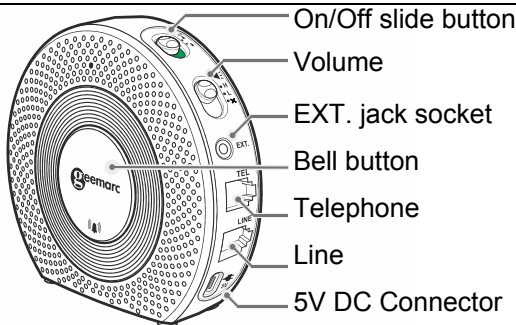

BATTERIES INSTALLATION

Your AMPLICALL™110 works with two Alkaline AA 1.5V batteries (not provided).

Please follow the below procedure to insert the batteries.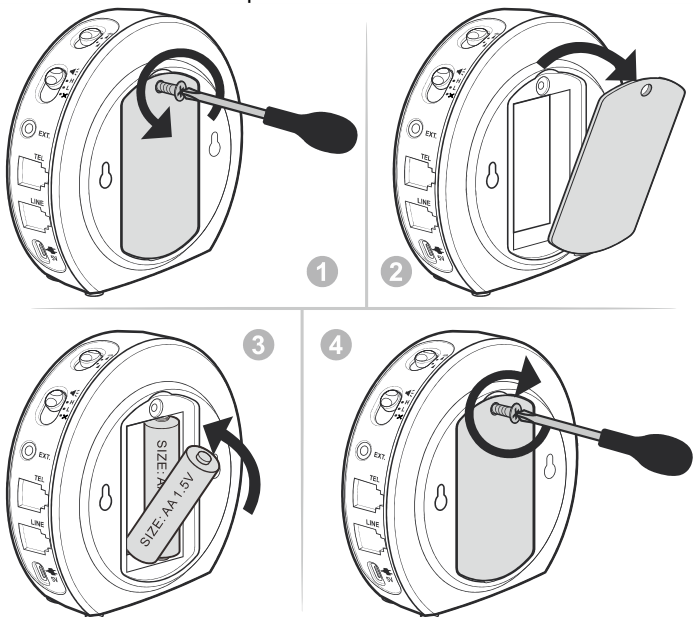


1. Unscrew the screw on the battery cover.
2. Open the battery cover.
3. Insert the 2 AA batteries into the compartment.
4. Close the battery cover and fix it using the screw.

Low Battery

If your device is running low on battery or if no battery has been inserted, your AMPLICALL™110 will send a signal to the receiver which will make the zone icon light up.

Connecting your AC110 to the mains (Optional)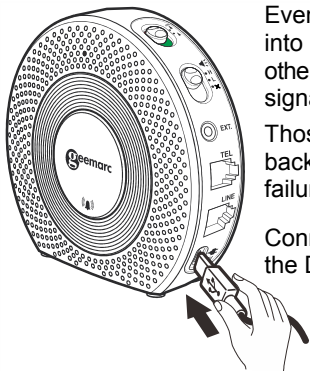

Even if your AMPLICALL™110 is plugged into the mains, you must insert batteries, otherwise your device will send a low battery signal to your receiver.

Those batteries will however only work as backup batteries in the event of a power failure.

Connect the type C USB power cable to the DC input connector.


Pairing your AC110 with the AMPLICALL receivers

The AC110 can work on its own, it will amplify the ringer and will flash when you receive a call. You also have the possibility to register your AMPLICALL™110 with the AMPLICALL receivers. If paired, the receivers will receive the information from the AMPLICALL™110 and ring and flash as well.

Enter pairing mode for the AMPLICALL receivers.

1. Press & hold the **LEARNING** button (●) for 3 seconds. The LED in zone icon #1 will flash
2. Press the **LEARNING** button (●) again to move to the zone icon you wish to pair the AC110 with.
Press the **melody selection** (♪) button to scroll through the melodies. Once you hear a melody you like, do not press any button during the melody.
3. Press once the bell button of the AC110 to pair.
4. If successful, the AMPLICALL receivers will generate two short tones.


Connecting your AC110 to a phone socket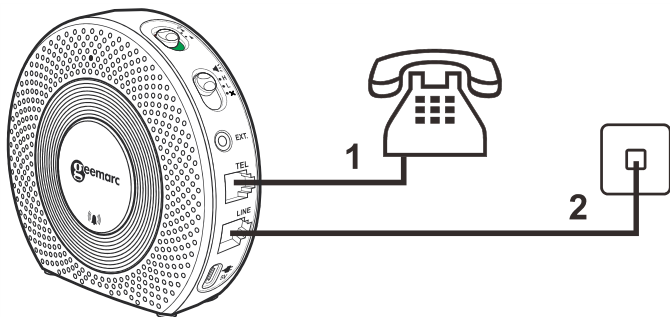

1. Connect one end of the supplied **RJ11 cord** to your phone socket and the other to the “**Tel**” input connector on your AC110
2. Connect one end of your original phone cord to the “**Line**” input connector on your AC110 and the other to your wall outlet.


Wall mounting

The AC110 can either be placed on a flat surface or be mounted on a wall.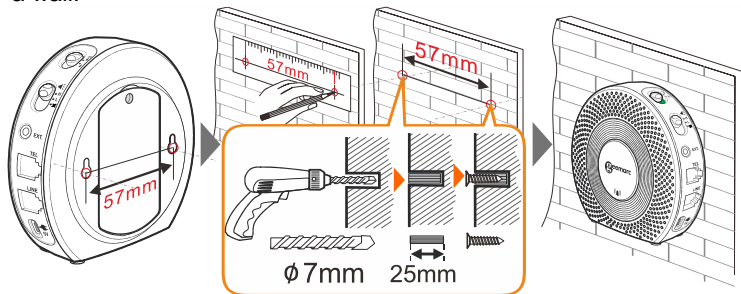

- Mark two holes on the wall using a pencil and drill hole template.
Mount the unit to the wall using the provided anchors and screws.


Charging the settings of your AC110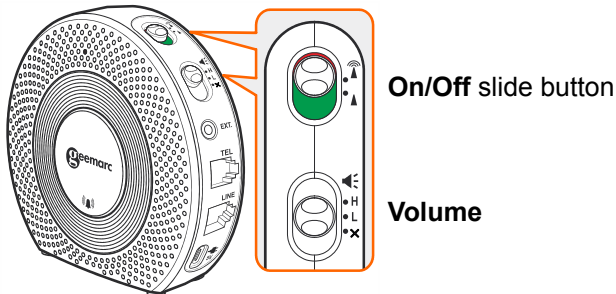

On (▲) / Off (▲) slide button: Slide to the **Off (▲)** position when no receiver is used.

Volume (▲): You can adjust the volume of the ringer by sliding this button, you can switch between the **H** (high), **L** (low), and **X** (silent) position.

Connecting an External Sensor to your AC110

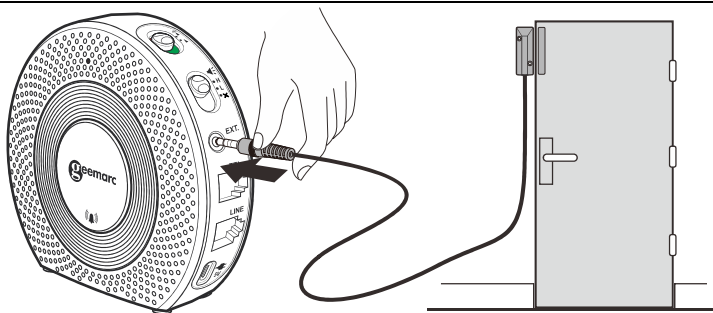The AC110 is compatible with external devices such as fall mats and other types of sensors which can be connected to the AC110 using the “**EXT**” jack socket. Once connected, the device will send a signal to the receiver every time it is triggered.

GUARANTEE

From the moment your Geemarc AMPLICALL™110 is purchased, Geemarc guarantee is for the period of two years. During this time, all repairs or replacements (at our discretion) are free of charge. Should you experience a problem then contact our help line or visit our website at www.geemarc.com. The guarantee does not cover accidents, negligence or breakages to any parts. The product must not be tampered with or taken apart by anyone who is not an authorized Geemarc representative. The Geemarc guarantee in no way limits your legal rights.

IMPORTANT : YOUR RECEIPT IS PART OF YOUR GUARANTEE AND MUST BE RETAINED AND PRODUCED IN THE EVENT OF A WARRANTY CLAIM.

Please note: The guarantee applies to the United Kingdom only.

DECLARATIONS:

UKCA Directives

Geemarc Telecom SA hereby declares that this AMPLICALL™110 is in compliance with the essential requirements and other relevant provisions of the Radio Equipment Directive 2014/53/UE.

The UKCA declaration of conformity may be consulted at www.geemarc.com

RF power : < 250mW

Recycling Directives

The WEEE (Waste Electrical and Electronic Equipment) has been put in place for the products at the end of their useful life are recycled in the best way.

When this product is finished with, please do not put it in your domestic waste bin. Please use one of the following disposal options:

- Remove the batteries and deposit them in an appropriate WEEE skip. Deposit the product in an appropriate WEEE skip.
- Or, hand the old product to the retailer. If you purchase a new one, they should accept it. Thus if you respect these instructions you ensure human health and environmental protection.

For product support and help visit our website at

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This product is made in China To Geemarc Telecom

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