Instruction Manual  
Manuale  
Manual  
Manuel d'utilisation  
Bedienungsanleitung  
Handleiding

CLARITY™  
Amplifying Your Life™

**Importer:**  
Plantronics B.V.  
Southpoint Building C  
Scorpius 140  
2132 LR Hoofddorp  
The Netherlands

© 2003 Clarity™, a Division of Plantronics, Inc.  
All rights reserved. Clarity, Clarity Power  
and Amplifying Your Life are trademarks or  
registered trademarks of Plantronics, Inc.

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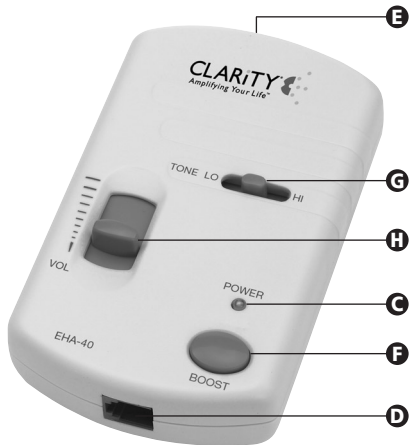
0800 242500

[www.clarityproducts.com](http://www.clarityproducts.com)

# Introduction to the Clarity™ Amplifier EHA40

Thank you for choosing the Clarity™ Amplifier EHA40. Before you use this product, we advise you to read this instruction manual thoroughly. You will then be familiar with all the functions, capabilities and safety instructions of this amplifier.

A 'quick reference' is incorporated into this manual. This is primarily aimed at the installation and setting of various functions. If you wish to use it, read only the highlighted parts of the text.

Keep this manual near your telephone, so that you can refer to it quickly if necessary.

GB

## Service and warranty

### Check the contents of the package

Check that the package contains all illustrated products. If this is not the case, contact the retailer where you bought this amplifier.

### Warranty

- The following 2 year warranty and service information applies only to product purchased and used in the countries listed on page 61 You may also contact your local retail dealer for warranty information in your country.
- Clarity™ guarantees the satisfactory condition of the equipment with regard to manufacturing and material defects for 2 years from the date of purchase. (Please keep your original receipt safely).
- This product must be installed and used in accordance with the instructions contained in this User Guide, and serviced at an approved Plantronics Service Centre. Failure to comply with these conditions may render the warranty invalid.

### 2 Year Warranty and Repair Service

- Products under warranty are replaced free of charge.
- All units serviced to full production standards using original factory parts.

### Customer Service

If you need support in installing or using your Amplifier EHA40, then please contact the Clarity Customer Service (see page 61).

## Safety Instructions

Before you use this amplifier, please read the following safety instructions. Please adhere to them strictly to avoid the risk of fire, electric shocks and personal injuries:

- Read all instructions carefully.
- Make note of the warnings and instructions on the product.
- Remove the plug of the telephone from the wall socket before you clean the amplifier.
- Do not use a liquid cleaning detergent or spray can, only a damp cloth.
- Do not use the telephone with the amplifier near water.
- Use the telephone flex and batteries as indicated in this instruction manual. Take empty batteries to a collection point for chemical waste.
- Never throw batteries into a fire; they might explode.

The Clarity™ EHA40 Amplifier complies with the following EC Directives:

- 73/23/EEC Low Voltage Directive.
- 89/336/EEC Electromagnetic Compatibility Directive.
- 93/42/EEC Medical Devices Directive (class 1).

## Parts of the amplifier

- A** Battery cover
- B** Battery
- C** Battery indicator (red)
- D** Connection to telephone handset
- E** Connection to telephone
- F** Boost button (for additional amplification)
- G** Tone control
- H** Volume control

## Connecting the amplifier

### Installing or replacing batteries (see **A**, **B** and **G**)

1. When you install or replace batteries, pull the outlet plug of your telephone out of the outlet and disconnect the Clarity™ Amplifier EHA40 from the telephone .
2. Press the catch of battery cover **A** upwards and pull it towards you. Then place the batteries as indicated in **B**.
3. Then press battery cover **A** properly closed.
4. If the batteries are flat, the red light **C** will not illuminate
5. Replace the batteries as soon as possible.  
The indicator will only illuminate if the amplifier is in use.

N.B. You need one 9 volt battery for the Clarity™ Amplifier EHA40. Battery life ranges from two to six months, depending on the frequency of use of the amplifier and the type of battery. On purchase, we recommend alkaline batteries.

### Checking the battery charge

You can measure battery capacity in the interim. With the handset on the base, press boost button **F**. If the indicator illuminates, you have sufficient battery capacity.

### Attention!

The red light **C** goes out gradually as soon as you replace the handset on the base. If the light does not go out, the handset has not been replaced correctly.

## Connecting the telephone handset (see ①)

1. Disconnect the telephone flex with handset from your telephone by removing the modular plug from the telephone.
2. Connect the handset to the amplifier.  
Connect the telephone flex via the opening at the bottom of the amplifier ①.

## Connecting the amplifier to the telephone (see ②)

1. Plug in the connection flex ② with one end in the top of the amplifier ② and the other end in the telephone. (where the handset was previously connected).
2. Pick up the handset and check whether you hear a dial tone.

The Clarity™ Amplifier EHA40 cannot be connected to a telephone with the dialling mode installed in the handset.

## Using the amplifier

### Boost Function (see ③)

The Clarity™ Amplifier EHA40 amplifies incoming sound up to 15 decibels as a standard. If you press the Boost button ③, the EHA40 will provide additional amplification up to 40 decibels.

Press and hold down the boost button for more amplification. It also eliminates any background noise and feedback by muting the microphone in your handset. You must remember to release the button when you speak so that the other party can hear you, then press and hold the boost button again when they speak.

Therefore:

- when you speak RELEASE the boost button.
- when you listen PRESS the boost button, for extra amplification.

The boost function switches off automatically when you place the handset back on the base.

### Tone control (see ④)

Use the tone button ④ to control the tone of the incoming voice. To increase the volume of the high tones, slide the button to the right. To reduce the volume again, slide the button to the left.

## Volume control (see )

The volume control button  enables you to adjust the volume of the voice of the person at the other end of the line. If you want to increase the volume, slide the button upwards.

To lower the volume, slide the button downwards.

**Clarity Power™ technology** ensures that only the high tones are amplified. This technology not only makes calls louder, but also easier to understand.

### Warning:

If the Boost function  is switched on and/or the volume control  is not at the 'normal' position the volume may be extra loud and in excess of 118dB.

This mode is intended specifically for hearing impaired users and could be harmful to other users. The volume should be increased gradually to find the minimum level of amplification that is comfortable for you. Avoid prolonged or unnecessary exposure to high volume levels.

## Problem Solving (solving problems yourself)

If you have problems, always check first whether:

- 1) all flexes are connected correctly;
- 2) the batteries are installed correctly or whether they need to be replaced.

To check whether the problems are caused by the amplifier, plug the telephone handset directly into the telephone again, without the amplifier in between. If this solves the problem, it could be caused by the amplifier. If the problem is not solved, it may be due to the telephone or telephone line. In this case, contact the manufacturer of the telephone or your telephone company.

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