**TV 2400-1**

**TV 2500-1**

**TV 2410NL-1**

**TV 2510NL-1**

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**Getting started**

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**Aan de slag**

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Your TV 2xxx-1 / TV 2xxxNL-1 is an additional receiver for your TV 2400 / TV 2500 / TV 2410NL / TV 2510NL. After having inserted and charged the battery you will have to register it to the transmitter. There is no limit number of receivers to be registered to the TV 2400 / TV 2500 / TV 2410NL / TV 2510NL transmitter for use together.

For more information about the operation and hints on safety and disposal, please refer to the manual of your TV 2400 / TV 2500 / TV 2410NL / TV 2510NL.

## Setting up the receiver

### Inserting and charging the rechargeable battery

- 1 Insert a battery into the headset receiver.
- 2 Charge the battery before first use. Switch off the receiver (position OFF). Place the receiver into the charging cradle of the transmitter. The charging process starts automatically. The battery LED indicator will be red and will turn to green when the battery is fully charged. This takes about 2 hours and 30 minutes.  
Alternatively you can use the fully charged spare battery from the transmitter.

### Registering the receiver

- 1 Turn off the transmitter if it is on (disconnect from the mains).
- 2 Disconnect the audio source input.
- 3 Turn on the NEW headset, the LED indicator will be flashing on and off slowly.
- 4 Press the MIC button and hold for 3s, the green LED indicator will flash on and off quickly.
- 5 Turn on the transmitter immediately (must be done within 9s, or need to redo step 4).
- 6 If the registration is successful, you will hear a short confirmation tone in the headset and the green light on the headset will automatically stop flashing and will light constantly. This indicates that registration was successful.
- 7 Reconnect the audio source input.

**Helpline UK 0844 800 6503**

(See user manual TV 2400/TV 2500/TV 2410NL/TV 2510NL for costs and hours of operation)

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Audioline GmbH, D-41460 Neuss